

Oxygen Carry Regulatory Fittings Installation

Rowland Emergency

Regulator Set up

Current



Correct



Regulator Set up

Current



Correct



Hex Nipple 1/8" M x 1/8" M

1/8" 90 elbow

Installation Guide

- 1: Take off regulator assembly (circled in yellow)
2. Clean off Teflon tape from threads in regulator
3. Remove Hanson from 90 (circled in red)
4. Clean off Teflon tape from threads in Hanson
5. Apply new Teflon tape to Rowland 1/8" 90 elbow (NF-EL90-F2M2) & Hex Nipple 1/8" M x 1/8" M (NF-HNM2-M2). All male threads should have Teflon tape
6. Attach hex nipple to 90 and hand tighten
7. Screw assembly into regulator
8. Tighten with wrench (on 90). Securely tighten and stop when in-line with regulator body (as per picture).
9. Next, add Hanson and securely tighten
10. Leak test with soapy water