

## ROWLAND ZOLL DEFIBRILLATOR POLE MOUNTING GUIDE



### **BASELINE POSITION**

Red lever is horizontal /parallel with mounting surface.



### **LOCKING POSITION #1**

Slide ZOLL defibrillator across mounting surface until you hear a click and observe **locking position #1**.

Red lever should be angled down.



### **LOCKING POSITION #2**

Continue to slide ZOLL defib across mounting surface until you hear a second click and observe **locking position #2** (baseline position).

Red lever should be horizontal/ parallel with mounting surface.

